

# Mix2Vial™ filter transfer set instructions

## STEP 1



Place WFI vial on a level surface and hold firmly  
Pierce the WFI vial

## STEP 2



Pull out the package and invert WFI vial

## STEP 3



Hold the product vial firmly on a level surface  
Pierce the product vial (WFI flows automatically)  
For BIOSTATE/Prothrombinex-VF:  
Gently swirl until dissolved

## INCORRECT CONNECTION METHODS



## STEP 4



Unscrew Mix2Vial and WFI vial by turning  
For Beriplex NZ:  
Gently swirl until dissolved (do not shake)

## STEP 5



Draw reconstituted product into syringe

WFI = Water for Injection vial.

For detailed instructions on reconstitution and administration, see package insert or Full Product Information.

These instructions are for  
the Mix2Vial™ filter transfer set  
provided with Prothrombinex®-VF,  
BIOSTATE® and Beriplex® NZ.

**Always read the label and use as directed.**

© BIOSTATE, Prothrombinex-VF and Beriplex® NZ are registered trade marks of CSL Limited.

For information: CSL Behring (Australia) Pty Ltd: For Medical/Technical Inquiries: Phone 0800 640 677.

For Customer Service Inquiries: Phone +61 3 9246 5231. E-mail: [customerservice@cslbehring.com.au](mailto:customerservice@cslbehring.com.au). Internet: [www.cslbehring.com.au](http://www.cslbehring.com.au)  
or New Zealand Blood Service: 71 Great South Road, Epsom, Auckland, New Zealand. Phone: 09 523 5744, Internet: [www.nzblood.co.nz](http://www.nzblood.co.nz).